- GB Toilet seat raiser**
- D Toilettensitzerhöhung**
- DK Toilet forhøjer**
- I Alza-water**
- S Toalettförhöjning**

*Item no.* **301102, 301104, 301106** (without lid)  
*Art. Nr.* **301101, 301103, 301105** (with lid)  
*Varenr.*  
*Codice*

## User manual

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

## **GB** Toilet seat raiser

### **Description & Intended Use**

The Mobilex Toilet Seat Raiser is medical device which intended use is to give stable and easy to reach seating support while disabled persons use the toilet. Any other use than intended is prohibited

### **Important Safety Information**

Use device only in good technical condition. Do not overload the device. Use the device only for purpose that it is intended for. Don't step on the device with your foot.

### **Set Up & Usage Description**

Loosen plastic wing nuts on both sides of the device. Lift the existing toilet lid and place the device on toilet so it lies flat and stable. The inner bottom collar should be located inside the toilet hole. Tie both wing nuts so the device is locked in position. Ensure that the device is locked and stable before each use.

### **Cleaning & Maintenance**

Clean the device regularly & keep it clean. Use soft cloth, warm water and neutral household washing agents. Do not use abrasives or bleach based cleaners.

The device is basically maintenance free. Regularly check it's rigidity & stability on the toilet. Don't use damaged or broken device.

**Recommended service life:** Normally 10 years depending on use.

### **Reuse**

The product is suitable for recycling.

### **Disposal of the product**

The product may not be disposed of with household waste but must be brought to the local recycling site.

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### Gurantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

### Declaration of Conformity (CE)

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the

### Mobilex toilet seat raiser

**Item no. 301101, 301102, 301103, 301104, 301105 and 301106**

(Artikel Nr., Varenr., Item no., Codice, varenr.)

### is designed and produced in accordance with the technical documentation and the basic requirements of the:

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### In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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